



Aerosol SPA system

Pneumatic Purging-Mist Mode


Instruction Use For Co-Care Nasal Irrigator



Components:

- a. Mist Nozzle
- b. Orifice for Mist Operating Valve
- c. Irrigation Cover
- d. Irrigation Cover O-Ring Seal
- e. Saline Solution Tank
- f. Saline solution Siphon Tubing
- g. Tubing
- h. Pressure Connector

How To Install & Use: (Refer to Left Figure)

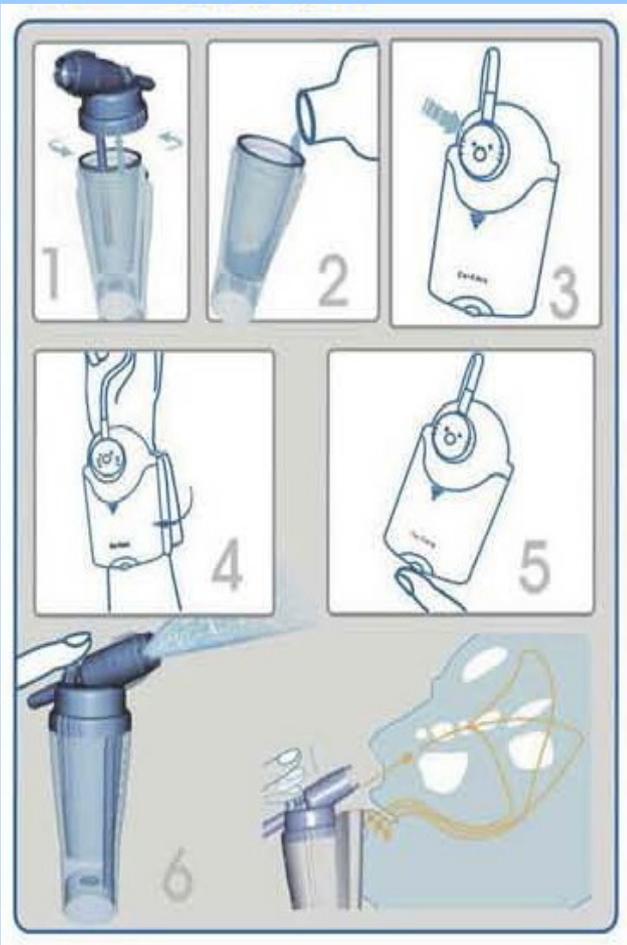

1. Unscrew the irrigation cover on saline solution tank to open.
2. Pour saline solution into the saline solution tank (20ml Max), re-screw the irrigation cover on saline solution tank tightly.
3. Connect the pressure connector onto the connector with O-ring seal and orifice for air inlet/outlet of vacuum pump / compressor and make sure they are connected tightly.
4. Put on wrist holder on the inside of user's wrist.
5. Switch on the power.
6. Position the mist nozzle near (0-5mm) to nostril, press the orifice for mist operating valve by single finger to start the rinsing work.

★ To avoid the injury resulted by choking, do not lay down while using the Co-Care.

Cleaning And Maintenance:

- 1. Let the nozzle keep spray ten sec. or more fully evacuate the solution and avoid the nozzle being blocked by remaining crystal salt.**
- 2. Switch off the power then remove the pressure connector from connector with O-ring seal and orifice for air inlet/outlet of vacuum pump /compressor .**
- 3. Discard all unused saline solution then clean all components (except tubing) with purified water and put them at cool place 1-2 hours for drying.**
- 4. For further sterilization, to steam all components (except tubing) with high temperature or put the components into boiled water boiling for ten minutes.**
- 5. The tubing doesn't need to sterilize, but, to replace the tubing every 3-6 months is suggestion.**

Cautions:

- 1. The nasal irrigator is applicable for all aged users. But, for baby or younger children, the operation is suggested with the help of adult.**
- 2. The nasal irrigator is designed with non-pressure and constant spray, which is safely applicable while nasal congestion. User can swallow and spit out the solution during cleansing process.**
- 3. The saline solution for contact lens, which contains chemical materials, will cause uncomfortable feeling on nasal cavity.**
- 4. Besides special solution or normal saline solution, other salty water is not allowed to avoid the block on mist nozzle.**
- 5. One set of components for one user only. Do not share the components with other user to prevent transmitting flu. virus infection,**
- 6. For keeping best effect on operation, to replace the components every 3-6 months. is a suggestion.**
- 7. Do not self-detach the irrigation cover or mist nozzle. Such action will cause mist nozzle block or irrigator damage.**
- 8. When spray is seriously blocked or unsuccessful, please contact the store where you brought it for repair. Do not use any tool for self-repair.**